



De Mayo Hip Positioner® System OBESE (Large Patient) Set Up Instructions

1. Leave OR table pads in place, then place OR bedsheet on top.




2. Pre-Assemble the Positioner. Assemble flat bar into bottom groove of Lumbar Support Base. Assemble washer and locking knob.



3. Roll the patient on their side.



4. Attach Bariatric Extension Plate (1101-BE).

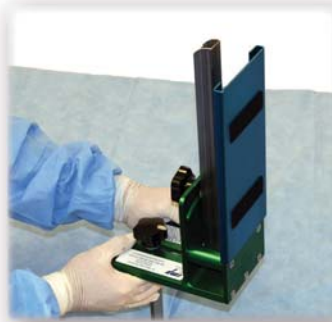

5. Attach Contoured Lumbar Pad for left or right hip application.




De Mayo Hip Positioner® System

OBESE (Large Patient) Set Up Instructions page 2


6. Assemble Lumbar Support Base into the imp® Rail Clamp at a position even with the patient's iliac crest. Use the Vertical Bar in a straight and upright position as a guide to aim at the patient's iliac crest.



Lock Flat Bar Knob in center groove of Support Base.

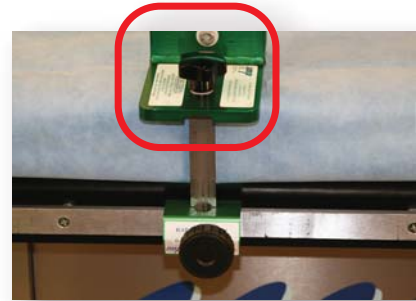

7. Angle the Vertical Support Bar towards the head of the patient. Assemble the Horizontal Crossover Large Bar #1 onto the Vertical Bar #1. Allow 2 fingerbreadths between the patient and the bar.

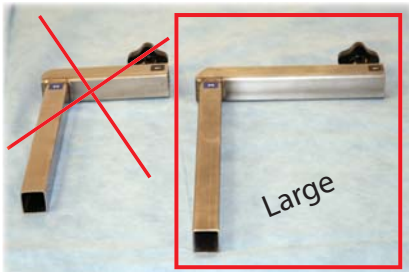


* Anterior Support Assembly (includes Anterior Support, Anterior Vertical Support -Large, Anterior Pelvic Support)



CAUTION: Make sure the threaded knob fits into groove on #3

Attach Anterior Rotation Bar #3 to round end of Horizontal Bar #3/2.

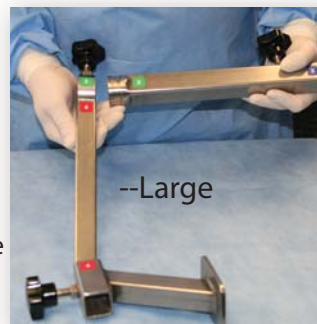

Attach Anterior Pelvic Support #4 to Anterior Vertical Bar #4.



De Mayo Hip Positioner® System OBESE (Large Patient) Set Up Instructions page 3

8. Place Anterior Patient Protector Pad



onto the Anterior Support Assembly.

9. Slide the "Large" Anterior Support Assembly over Crossover Bar #2. Rotate Anterior Vertical Bar #3, and adjust Anterior Pelvic Support #4, so that the Anterior Patient Protector Pad contacts the superior iliac crest.



10. Compress Anterior Support Assembly pushing toward Lumbar Support.



Lock Anterior Support Knob #2.



TIGHTEN ALL KNOBS FOR SECURITY



If any readjustments are required, release Compression Knob #2, then adjust any other knobs to achieve greatest stability.

De Mayo Hip Positioner® System OBESE (Large Patient) Set Up Instructions page 4

OPTION - Stabilizer Bar may be inserted into the Anterior Support Assembly with an additional imp® Rail Clamp (note: maintain 2 fingerbreadths at the rib cage and Horizontal Crossover Bar).

Add the Stabilizing Bar Tube by sliding it from the anterior side of the patient into the opening of the Anterior Support Assembly.



Slide the Stabilizing Bar Hook into the imp® Rail Clamp. Attach to the threads of the Stabilizing Bar Tube, and tighten both knobs.

In the event (while utilizing the Stabilizing Bar) you cannot seat the Adjustable Support Assembly, loosen compression and rotate plate. With the Anterior Pelvic Support out of the way, seat the Adjustable Lateral Support and re-engage the Anterior Pelvic Support.



imp® Innovative
Medical Products, Inc.

A GLOBAL LEADER IN PATIENT POSITIONING

87 Spring Lane, Plainville, CT 06062

PH: 860-793-0391 FAX: 860-793-8975

Toll Free PH: 800-467-4944 FAX: 888-229-1452

www.impmedical.com

De Mayo Hip Positioner® is a registered trademark of Innovative Medical Products, Inc.

De Mayo Hip Positioner® US Patent #6,820,621

imp® Rail Clamp is a trademark of Innovative Medical Products, Inc.

© 2011 Innovative Medical Products, Inc.

All Rights Reserved