

imp® LapWrap® Positioning Pad Set Up Instructions



1. Place LapWrap® on OR table pads with foam side down, and VELCRO® brand hook-and-loop fastener side UP.

2. Place patient so the shoulder lies above the upper edge of the LapWrap®.



Lift LapWrap® center tongue of the upper arm and pull between the torso and the biceps.

3. Pull the tongue out through the cutout, while pulling the VELCRO® brand hook-and-loop fastener around the outer arm and towards the body. Fasten to the center tongue.



Optional Technique: Wrap entire 3 piece strap around the patient's arm and attach VELCRO® brand hook-and-loop fastener near or under patient.

LapWrap® Positioning Pad Set Up Instructions pg.2



4. Repeat for forearm placement.



Optional Technique: Place TUFFease® Gel redistribution pad on OR table for extra patient protection.



Universal Design Fits All Size Patients



Ideal for various robotic laparoscopic & bariatric surgeries

Always use appropriate positioning safety precautions



Trendelenburg



reverse Trendelenburg



Latex Free

Our product features genuine VELCRO® brand fasteners.

LapWrap® is a registered trademark of Innovative Medical Products, Inc.

LapWrap® - PATENT APPLIED FOR

TUFFease® is a registered trademark of Innovative Medical Products, Inc.

VELCRO® is a registered trademark of Velcro Industries B. V.

© 2010 Innovative Medical Products, Inc. ALL RIGHTS RESERVED

Rev 10/2010

imp® **Innovative**
Medical Products, Inc.

87 Spring Lane, PO Box 8028

Plainville, CT 06062

PH: (860) 793-0391 FAX: (860) 793-8975

Toll Free PH: 1-800-467-4944 FAX: 1-888-229-1452

www.impmedical.com