

Phase 4 Gel™ Splint

STERILE DRAPING TECHNIQUE UTILIZING HOSPITAL SUPPLIED DRAPES AND BARRIERS



① Prep the patient's arm



② Exchange the sterile splint to the field



③ Remove the protective plastic sheet from the Gel



④ Insert the operative hand, aligning the thumb in the cut out of the universal splint

Phase 4 Gel™ Splint

STERILE DRAPING TECHNIQUE UTILIZING HOSPITAL SUPPLIED DRAPES AND BARRIERS



5 Connect closures over the fingers and forearm securing the operative arm

6 Circulator attaches the Positioner Hook to the sterile splint



7 Proceed with hospital draping protocol

Ordering Information

Cat. # 509-P4

10 /Case

* ILLUSTRATION ONLY -HOOK(S) NOT INCLUDED



Phase 4 Gel™ is a trademark of Innovative Medical Products, Inc.
Phase 4 Gel™ Splint -PATENT APPLIED FOR
Surgical Photos courtesy of Alan Reznik, M.D. New Haven CT
© October 2011 All Rights Reserved

imp Innovative
Medical Products, Inc.

87 Spring Lane, PO Box 8028
Plainville, CT 06062

PH: (860) 793-0391 FAX: (866) 459-1805

Toll Free PH: 1-800-467-4944 FAX: 1-888-229-1452

www.impmedical.com