



# Reznik Universal Shoulder Positioner™

US Patent # 7,569,024

## Set Up Instructions



1. Position Rail Clamp and Positioner on the side where the patient is facing.



Insert Positioner into Rail Clamp and rotate Positioner so the Large Lever No. 1 is facing the operator.

Release Lever No. 1 to adjust horizontal bar.

2. Release both Vertical Adjusting Levers No. 2 to achieve appropriate height of Horizontal Bar.




# Reznik Universal Shoulder Positioner™ pg. 2

US Patent # 7,569,024

## 3. Rotate Vertical Bar so

a.) Head Swivel Pulley faces patient's head, b.) Traction Swivel faces foot of table

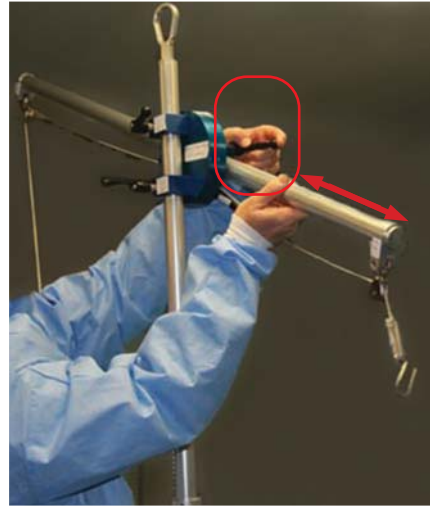

Release Horizontal Adjustment Lever No. 3 for horizontal length adjustment.


## 4. Lock knob on Rail Clamp and check locking of all levers. Lock all knobs and levers securely.



After Sterile Phase 4 Gel™ Splint is applied to patient's forearm, secure hook to forearm splint.



## 5. Apply traction weight to opposite hook, per the surgeon's preference.

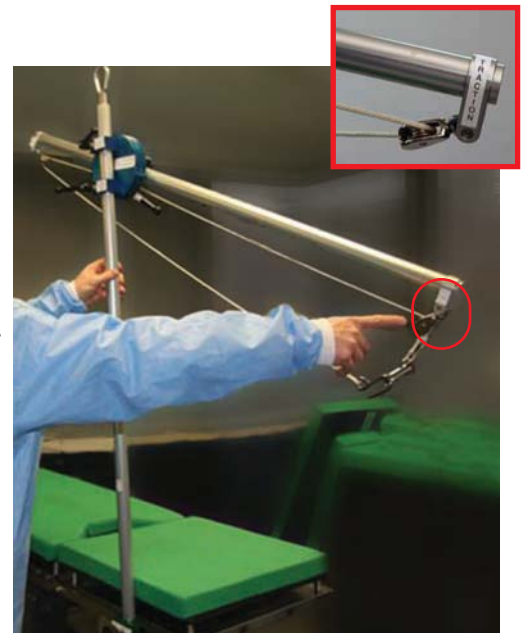

**imp** Innovative  
Medical Products, Inc.  
87 Spring Lane, PO Box 8028  
Plainville, CT 06062  
PH: (860) 793-0391 FAX: (866) 459-1805  
Toll Free PH: 1-800-467-4944 FAX: 1-888-229-1452  
[www.impmedical.com](http://www.impmedical.com)