



GUIDE WIRES 198GW-NSP CLEANING & STERILIZATION PROCEDURE

NOTE: ALL SOLUTIONS MUST BE COMPATIBLE WITH ALUMINUM & STAINLESS STEEL.

Recommended Hand Cleaning Instructions:

- Rinse the product in warm tap water.
- Wash the product with an instrument detergent up to a pH of 9.0 or enzyme product.
- RINSE THE PRODUCT IN WARM TAP WATER.
- DRY THOROUGHLY AND WRAP.

Recommended Sterilization Instructions:

- Insure that all parts are thoroughly cleaned.

Pre-Vac Sterilization

Double wrap in two disposable or reusable wraps.

Run normal vacuum cycle for your institution.

Minimum recommended parameters are as follows:

EXPOSURE TIME:	4 MINS
TEMPERATURE:	270* - 275* F 133* - 135* C
DRYING TIME:	20 MINUTES
COOLING:	@ ROOM TEMPERATURE