
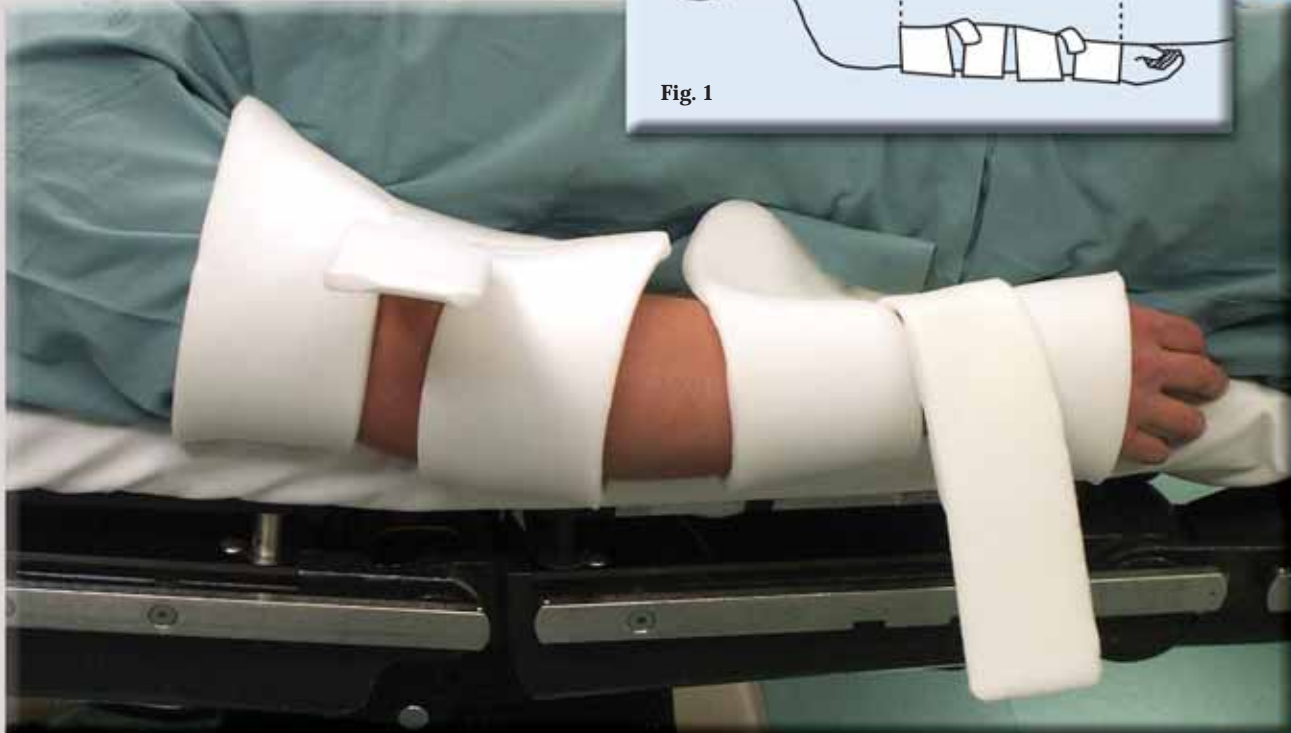


imp LAPWRAP™ POSITIONING PAD

"For me, securing a patient's arms by their sides has always been a real pain. Since using the Lapwrap, it has become a piece of cake. I like the ease of use, the protection it affords and the fact that I can access the patient's arms during the case without interrupting surgery. When I have the need to tuck the patient's arm, one of the first things I do is make sure the Lapwrap is on the OR table. My only complaint is that it was not available when I first went into practice!"

Thom L. Holt, CRNA
Conway, South Carolina

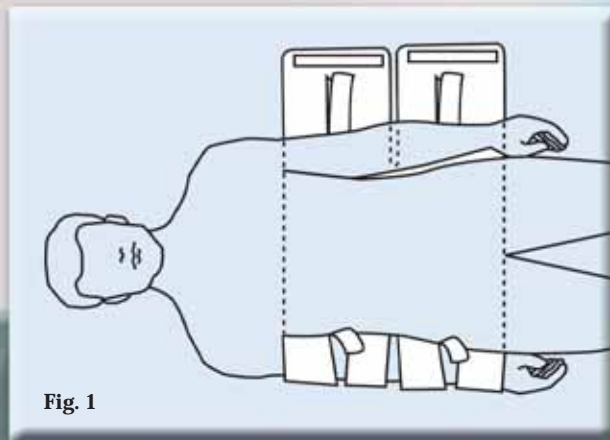

- Secures patients' arms by their sides
- Holds IV tubes and Leads for anesthesia
- Protects neurological structures of the elbow
- Acts as a warming blanket for patient
- Simple – Latex-free – Disposable
- For laparoscopic, head/neck and orthopedic surgery

"The LapWrap works great for my cervical spine cases. It used to be a hassle tucking the arms and this product makes it simple. I am not concerned about nerve palsy problems with the soft foam padding protecting the arms. I've asked to have it used on all of my cervical spine cases."

William Mills, MD (spine surgeon)


"I wondered when someone would make a product so easy to use like the LapWrap. I have put it on my needs card for all my laparoscopic cases where the arms need to be tucked."

Terry Levenson, MD, (OB/GYN)

INSTRUCTIONS:

1. Place LapWrap on O.R. Table with Velcro up.
2. Place patient so the shoulder lies above the upper edge of the LapWrap. (Fig. 1)
3. Lift LapWrap center tongue of the upper arm and slide between the torso and the biceps. Push the tongue out thru the cutout while pulling the velcro around the outer arm and towards the body and fasten to the center piece or center tongue.
4. Repeat the above step for the forearm placement.

The LapWrap can be trimmed with scissors for better viewing of wire leads, tubing, or connections.

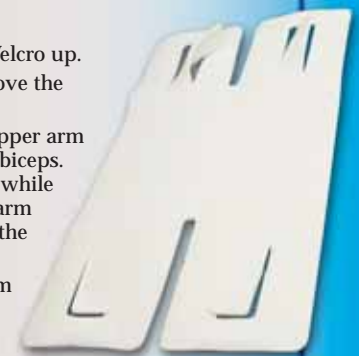

Cat. # 904
Patent Applied For

imp Innovative
Medical Products, Inc.

A GLOBAL LEADER IN PATIENT POSITIONING

87 Spring Lane, Plainville, CT 06062

PH: 860-793-0391 FAX: 860-793-8975

Toll Free: PH: 800-467-4944 FAX: 888-229-1452

www.impmedical.com